

Packaging MDL ThinPrep Specimens for Local MDL Driver Pick-up.

1

Insert labeled vial into the tray.



Insert the tray into Ziplock bag.

2

3

Insert the sealed bag with tray into the mailer.



Be sure to include a completed test requisition form for each specimen in the top of the mailer.

4

5

Place specimens in your MDL Specimen Kit Lock box for MDL driver pick-up.



Medical Diagnostic Laboratories
www.mdlab.com • 877.269.0090



Packaging MDL ThinPrep Specimens for Local MDL Driver Pick-up.

1 Insert labeled vial into the tray.



2 Insert the tray into Ziplock bag.



3 Insert the sealed bag with tray into the mailer.



4 Be sure to include a completed test requisition form for each specimen in the top of the mailer.



5 Place specimens in your MDL Specimen Lock box for MDL driver pick-up.



Contact us at 877 269 0090 · www.mdlab.com